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Liebe Anwender!

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Patient Information Leaflet

Dear Patient!

Please read the following Patient Information Leaflet carefully as it contains important information, which you should be aware of when using this medical device. Please consult your doctor, pharmacist, optician or nonmedical practitioner if you have any questions.

STERILE A

Lento Nit® K Eye Drops CE 0482

Composition

10ml isotonic solution contains:
Potassium iodide 50mg,
Calcium chloride 2 H₂O 49mg,
Sodium thiosulphate 5 H₂O 5mg,
Chlorhexidine gluconate 0,65mg
(Preservatives),
Sorbitol, water for injection

Pharmaceutical form and contents

Lento Nit® K Eye Drops are available in 10ml packs and multipacks containing 3 x 10ml.

Lento Nit® K Eye Drops is a medical device for reducing metaplastic changes in the eyes, which are associated with impaired vision due to clouding. In addition, strained and tired eyes are supplied with nutrients and vital salts.

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Therapeutic indications

The eye drops are for nourishing and moistening the eye as well as for preventing clouding of the crystalline lens and the vitreous body.

Contraindications

Lento Nit® K Eye Drops should not be used in the event of hyperthyroidism or hypersensitivity to iodine.

Lento Nit® K Eye Drops should only be used on children or during pregnancy and whilst breastfeeding under medical supervision.

Special warnings

There are no known effects of **Lento Nit® K Eye Drops**, which affect the ability to drive, operate machines or work without being able to stop safely. Please note that your vision may be impaired for a short time immediately after using the eye drops.

Interactions

No interactions have been observed. In principle, high doses of potassium iodide in conjunction with potassium-sparing diuretics (substances, which increase urination) could result in hyperkalaemia (increased serum potassium levels). Please note that this could also apply to medicines taken recently.

Dosage

Apply 1-2 drops of **Lento Nit® K Eye Drops** into the eye conjunctival sac three times per day.

Application errors

A single application of **Lento Nit® K Eye Drops** in a too large quantity is not expected to cause any effects.

Side effects

Side effects are not to be expected if this product is used according to the instructions. However, if you notice any side effects, please inform your doctor, pharmacist, nonmedical practitioner or optician. Irritation could occur in the event of hypersensitivity to one of the ingredients. If this happens, you should stop using the product.

Information on shelf-life and storage

The expiry date is printed on the exterior of the pack.

Do not use this pack after this date!

The eye drops can be used up to 4 weeks after opening.

Do not store above 25 °C.

Keep this medical device out of the reach of children.

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Optima Pharmazeutische GmbH