# Lansinoh®

## Manual Breast Pump

**INSTRUCTIONS FOR USE**

# Table of Contents

|                                                                      |    |
|----------------------------------------------------------------------|----|
| [EN / ENGLISH] Manual Breast Pump<br>Instructions for Use.....       | 1  |
| [FR / FRANÇAIS] Tire-Lait Manuel<br>Instructions d'utilisation.....  | 6  |
| [DE/ DEUTSCH] Lansinoh® Handmilchpumpe<br>Gebrauchsanweisung .....   | 11 |
| [NL/ NEDERLANDS] Handkolf<br>Gebruikshandleiding .....               | 16 |
| [TR/ TÜRKÇE] Manuel Göğüs Pompası<br>Kullanım Talimatları.....       | 21 |
| [PL / POLSKI] Laktator Manualny<br>Instrukcja użytkowania.....       | 26 |
| [HU / MAGYAR] 2 fázisú kézi mellszívó<br>Használati útmutató .....   | 31 |
| [RO / ROMÂNĂ] Pompă de sân manuală<br>Instrucțiuni de utilizare..... | 36 |

## ENGLISH

### CONTENTS

1. Introducing the Lansinoh® Manual Breast Pump
2. Lansinoh® Manual Breast Pump Parts
3. How to Sterilise and Clean Your Lansinoh® Manual Breast Pump
4. How to Assemble Your Lansinoh® Manual Breast Pump
5. How to Use Your Lansinoh® Manual Breast Pump
6. Breastmilk Storage Guidelines
7. Tips About Pumping
8. Troubleshooting for the Lansinoh® Manual Breast Pump
9. Feeding breastmilk with the NaturalWave® Peristaltic Teat

### Intended Use of Breast Pump

Lansinoh® Manual Breast Pump is intended to express and collect the breastmilk of a nursing woman for the purpose of feeding collected breastmilk to a baby. Lansinoh® Manual Breast Pump is intended for a single user.

### Contraindication of Breast Pump


There are no known contraindications.

## 1. Introducing the Lansinoh® Manual Breast Pump

The Lansinoh® Manual Breast Pump has been designed specifically with the needs of breastfeeding mothers in mind. This breast pump has two phases: Let-Down Phase and Expression Phase. You can easily switch between the two phases by adjusting the position of the handle on the stem. The ComfortFit® breast cushion creates a seal providing reliable suction resulting in comfortable and fast milk flow. The wide neck bottle is interchangeable with the entire Lansinoh® pump range. The pump is easy to assemble, easy to use and easy to clean.

## 2. Lansinoh® Manual Breast Pump Parts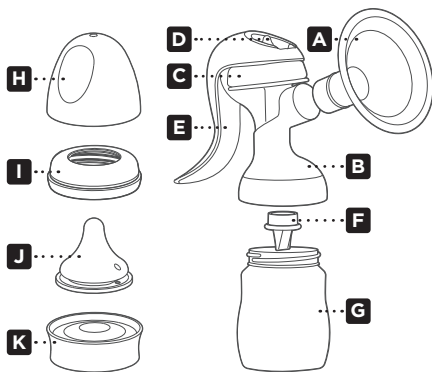

| NAME |                                       | ITEM NUMBER |
|------|---------------------------------------|-------------|
| A    | ComfortFit® Flange/<br>Breast Cushion | 53408       |
| B    | Manual Pump Body                      | 50631       |
| C    | Silicone Diaphragm                    | 50626       |
| D    | Manual Pump Stem                      | 50627       |
| E    | Manual Pump Handle                    | 50621       |
| F    | White Valve                           | 53480       |
| G    | Collection Bottle                     | 53562       |
| H    | Bottle Cap                            | 53462       |
| I    | Bottle Collar                         | 53463       |
| J    | NaturalWave® Nipple                   | 71027       |
| K    | Bottle Storage Lid                    | 53564       |

## 3. How to Clean and Sterilise Your Lansinoh® Manual Breast Pump

Prior to using your pump for the first time, disassemble it (remove silicone diaphragm and stem, ComfortFit® breast cushion and white valve) and boil all parts for 5 minutes. Remove all parts from water immediately after boiling. Completely air dry before assembly and before the next use.

**Note:** It is essential that you use enough water to keep the parts freely floating while boiling.

### Cleaning:




The ComfortFit® Breast Cushion, bottle, and handle may be cleaned in the upper compartment of your dishwasher or hand washed with dish soap and hot water. Do not use solvents or abrasives. Rinse with hot, clean water. The white valve, silicone diaphragm, stem, and the ComfortFit® breast cushion can be washed in the dishwasher; however, we recommend washing them by hand to prevent loss and to extend their usefulness. Completely air-dry all parts on a clean towel or drying rack.

**Note:** Use caution when cleaning the small white valve. It is important for proper suction. Do not use excessive force and do not clean with a baby bottle brush or a nipple brush. Gently wash by hand in hot, soapy water and rinse under hot running water.


## 4. How to Assemble Your Lansinoh® Manual Breast Pump

**Note:** Prior to assembling your Lansinoh® Manual Breast Pump, ensure that all parts have been cleaned and sterilised as detailed in section 3 of this manual.

1. Wash your hands thoroughly.
2. Insert the stem into the silicone diaphragm.
3. Place the silicone diaphragm and stem on top of the pump body. Make sure it fits securely around the rim by pressing down with your fingers to ensure a perfect seal.
4. Gently secure the white valve onto the pump from underneath, making sure the white valve is flush with the pump body. It is not necessary to use excess force when applying the white valve. If you find that the white valve is difficult to remove, you have pushed it too far onto the pump body.
5. Screw the bottle onto the pump body.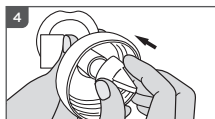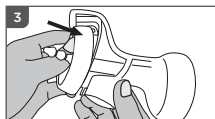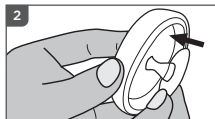

6. Align the groove in the handle with the stem and gently press the handle down. Push until you hear it click.

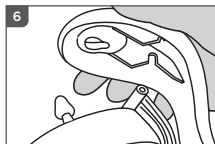
7. Adjust the stem and handle so that the handle fits the first notch of the stem, known as the Let-Down Phase.\*


\*This breast pump has two separate phases, Let-Down Phase and Expression Phase. You can easily switch between the two phases by adjusting the position of the handle on the stem.

8. Attach the ComfortFit® breast cushion to the pump body.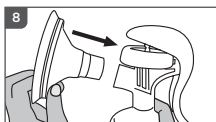

9. When fully assembled, this is what your pump should look like.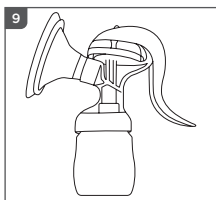

## 5. How to Use Your Lansinoh® Manual Breast Pump

If possible, wait until your breastmilk supply and breastfeeding schedule are well established. This is normally at least 3-4 weeks after birth, advised otherwise by your health professional.

1. Wash your hands thoroughly and make sure your breasts are clean.
2. Make sure you have sterilised and carefully assembled your pump exactly as described in the sections "How to Clean and Sterilise Your Lansinoh® Manual Breast Pump" and "How to Assemble Your Lansinoh® Manual Breast Pump."
3. Centre the pump opening over your nipple and gently press it against your breast ensuring that your nipple is centred in the tunnel of the breast cushion. Press the ComfortFit® breast cushion of the pump firmly against your breast making sure the seal is secure so that no air can escape. You can create a good seal by using your other hand to support your breast from underneath.

If your nipple is unable to be centred within the tunnel of the breast cushion without rubbing, you may need the larger size ComfortFit® breast cushion.

4. As you gently push down on the pump handle you will feel the suction on your breast. You do not need to depress the handle fully to create a vacuum, only as much as is comfortable.
5. You can create gentle suction pressure in preparation for expressing by setting the handle in the first (top) notch of the stem. Confirm that your nipple is moving and squeeze and release the handle until you find your preferred pace.


**Warning:** Do not continue pumping for more than five minutes at a time if no result is achieved. Try expressing at another time during the day. Please remember: Should pumping become very painful or uncomfortable, you should stop using the pump and consult with your health professional.

### Using Lansinoh's 2-Phase Technology

This breast pump has two phases, Let-Down Phase and Expression Phase. You can easily switch between the two phases by adjusting the position of the handle on the stem.

#### Let-Down Phase

This phase gently stimulates the breast. It is designed to mimic the way a baby feeds at the beginning of a feeding. It is one of the features that encourages milk let-down and encourages your milk to begin to flow. You can create gentle suction pressure by setting the handle in the first (top) notch in the stem and using quick pulses with the handle.
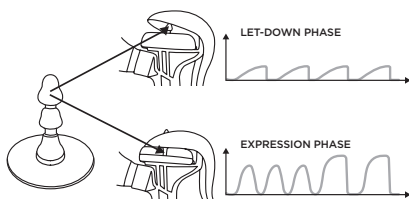


#### Expression Phase

This phase allows you to express breastmilk from the breast. Switch to this phase by setting the handle in the second (bottom) notch in the stem. To get the strongest suction fully depress the handle. For reduced suction strength depress the handle less. To start with the Let-Down Phase, set the stem in the first (top) notch of the handle and squeeze the handle. Your breastmilk will soon start flowing even though you may not use all the suction the pump can generate.

Once breastmilk is flowing (1-2 minutes) change to Expression Phase.

To switch to Expression Phase, gently press the "PUSH" part at the tip of the handle to set the handle to the second (bottom) notch of the stem. Continue to squeeze and release the handle to find the appropriate suction pressure. If the suction pressure feels weak when expressing your breastmilk, squeeze the handle all the way. The suction pressure changes based on how hard you squeeze the handle. Expressing in comfort is more important than setting a strong suction pressure. Please express milk at a suction pressure that feels appropriate for you.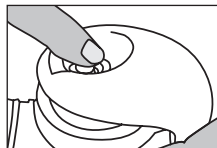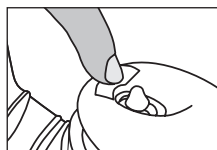

\*To return to Let-Down Phase from Expression Phase, (1) Pull the tip of the stem towards you, (2) gently squeeze the handle, and set it in the first (top) notch.

## 6. Breastmilk Storage Guidelines

### Breastmilk Storage and Thawing Guidelines for Healthy Term Babies

Check with your healthcare professional or an International Board Certified Lactation Consultant for specific storage instructions.

When freezing breastmilk, store in the back of the freezer. Do not store breastmilk in the freezer door as the temperature can fluctuate when the door is opened and closed.

Before storing, date all breastmilk and use the oldest breastmilk first.

#### Thawing Breastmilk

Thaw frozen milk in the refrigerator or place the containers under cool running water, gradually warming the water until the milk is thawed. Thawed milk can be kept in a refrigerator for up to 24 hours.

DO NOT refreeze milk once it is thawed.

Gently swirl thawed milk before feeding to mix the layers that have separated. Breastmilk is not homogenized and the "cream" will rise to the top. Breastmilk can vary in colour and consistency depending upon the time of the day that the milk was expressed, what you have eaten, and the age of the baby.

at the time of pumping. For further advice, please contact your health care professional.

**DO NOT microwave breastmilk!**  
Microwaving alters the composition of the milk. Microwaving also may create "hot spots" in the milk that could burn the infant.

**DO NOT thaw breastmilk in hot or boiling water.**

## 7. Tips About Pumping

- Familiarise yourself with the pump and parts prior to use.
- The most important part of successful pumping is the let-down reflex. This is when your breastmilk begins to flow freely. Warm wet compresses applied to the breasts and gentle, circular massaging of the breasts prior to and/or during a pumping session can help encourage this natural reflex.
- Try to choose a time to pump when you are not rushed and will not be interrupted. Trying to relax and thinking of your baby or looking at a photo of your baby can also be helpful when pumping your breastmilk.
- Overall a pumping session can last about 20-30 minutes but it depends on your body. Remember, successful pumping is a learned art. Early practice sessions may be shorter or longer than stated here and may result in a small amount of breastmilk collected, but that does not mean your body is not making enough milk or that you are doing anything wrong. Milk is produced on a supply and demand basis and pumping your breasts regularly in addition to nursing your baby stimulates and maintains your breastmilk supply.
- If breastfeeding problems occur, consult with your health professional. You may pump one breast while nursing the baby on the other. This allows you to maximize pumping effectiveness using the let-down reflex stimulated by your baby's suckling.
- Make sure the tunnel of the breast cushion is tilted slightly downward to allow your breastmilk to flow naturally toward the bottle.
- Do not turn the pump upside down after pumping.

### Precautions when feeding expressed breastmilk:

- **DO NOT MICROWAVE BREASTMILK!** Microwaving alters the composition of the milk and may also create "hot spots" in the milk that could burn the infant.
- Always test the temperature of any liquid before feeding your baby.
- Do not let a child take a bottle to bed or self-feed for long periods.
- To prevent possible choking hazard, test strength of the nipple by pulling on the bulb portion. Discontinue use if any tear or crack appears.

## 8. Troubleshooting for the Lansinoh® Manual Breast Pump

### Lack of Suction / Decreased Suction

If it feels like the suction has decreased, try the following:

- Make sure that all pump parts have been properly dried and properly assembled.
- Inspect the white valve for a tear or pinhole. This component is critical to achieving proper suction. If necessary, replace the white valve with the spare valve provided with the pump.
- Ensure that the silicone diaphragm and stem are fitted securely to the pump body and a perfect seal is created.
- Ensure that the ComfortFit® breast cushion is fitted securely to the flange body and a perfect seal is created.

- If your nipple is unable to be centred within the tunnel of the flange without rubbing, you may need a larger size ComfortFit® breast cushion.

### No Breastmilk Expressed

If you are not successful in expressing any milk, try the following:

- Make sure that all the parts have been properly dried and assembled and that proper suction is created.
- Try to relax as successful pumping is a learned art and may take several practice attempts to get started. Gentle breast massage before and during pumping can also aid in milk let-down and flow.
- Please refer to the "Tips About Pumping" section.

### Pain When Pumping

If you are experiencing pain when expressing breastmilk, try the following:

- You may be pumping too hard. It may not be necessary to use all of the suction that the pump can generate. Try depressing the handle half way.
- You may need a larger or smaller breast cushion. To order replacement parts or larger size breast cushions please call +4(0)113 259 1425 or visit [Lansinoh.co.uk](http://Lansinoh.co.uk) for details in the UK. You can also contact your country distributor.

## 9. Feeding breastmilk with the NaturalWave® Peristaltic Teat


**Instructions for Use:** Fill with the required amount of liquid and tighten the screw cap. Heating in a microwave is not recommended. If you choose to heat food in the microwave, do not seal the bottle. Remove teat, collar and cap when using in the microwave. Shake well before use and always check temperature before offering to baby. Unevenly heated food may burn baby's mouth. Do not microwave human milk as microwaving alters the composition of the milk. Before first use, clean the product. To ensure product hygiene, put the teat and the bottle with all its components in boiling water for 5 minutes before use. When carrying a full bottle, make sure it is closed with the cap in place, taking care to position the teat in the centre of the cap to activate the "spill-proof" function of the product.

**Cleaning:** Before initial use, place the product in boiling water for 5 minutes, allow to cool, and thoroughly rinse all bottle components. This is to ensure hygiene. Subsequently, the bottle can also be washed in the upper rack of a dishwasher and can be sterilised (open) both in the microwave and using a special sterilising machine. Always wash all components of the product immediately after each use. Wash in warm soapy water and rinse thoroughly. Clean before each use.

**Assembly:** Once thoroughly dry, insert feeding teat through the top of the screw cap collar. Attach the screw cap collar securely onto the bottle neck.

**Precautions:** When not in use, store in a dry and covered place. Always use this product with adult supervision. Do not clean, store or allow to come into contact with solvents or harsh chemicals. Damage could result. Do not leave the product in direct sunlight or heat or leave in disinfectant ("sterilising solution") for longer than recommended. Do not warm the liquid by placing the cup directly onto gas or electric rings, warming plates, in electric ovens etc.

**For your child's safety and health WARNING!** Always use this product with adult supervision. Never use feeding teats as a soother. Continuous and prolonged sucking of fluids will cause

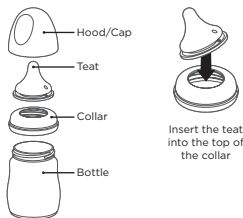tooth decay. Always check food temperature before feeding. Keep all components not in use out of reach of children. Do not leave the baby alone with drinking equipment due to risk of choking, the baby falling over or if the product has disassembled.

**WARNING:** Tooth decay in young children can occur even when non-sweetened fluids are used. This can occur if the baby is allowed to use the bottle/cup for long periods through the day and particularly during the night, when saliva flow is reduced or it used as a soother. Do not leave the teat in direct sunlight or heat, or leave in disinfectant ("sterilising solution") for longer than recommended, as this may weaken the teat. Inspect before each use. Throw away at the first sign of damage or weakens. Replace the teat every 7 weeks, for safety and hygiene reasons. Pull the feeding teat in all directions to check the product.

**ATTENTION:** Some juices are sterile and solutions may make the silicone become opaque. This does not modify any of the properties. Do not microwave human milk as microwaving alters the composition of the milk. Heating in a microwave oven may produce localised high temperatures. Take extra care when microwave heating. Always stir heated food to ensure even heat distribution and test the temperature before serving.

**WARNING:** Packaging materials are not part of the product. For your child's safety, please remove and discard all packaging materials prior to use but read and retain the instructions for future reference. Leave open in the microwave.

| SYMBOLS                                                                            | MEANING OF THE SYMBOLS                                                               |
|------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------|
|    | Read and follow the instructions for use                                             |
|    | Identifies the manufacturer                                                          |
|    | Indicates the date of manufacture                                                    |
|    | Indicates the part number of the device                                              |
|    | Indicates the batch number                                                           |
|    | Keep the device away from sunlight                                                   |
|   | Keep the device dry                                                                  |
|  | Indicates that the packaging contains products intended to come in contact with food |
|  | Indicates to handle the fragile device with care                                     |

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Lansinoh®, NaturalWave®, and ComfortFit® are registered trademarks of Lansinoh Laboratories, Inc.
© 2024 Lansinoh Laboratories, Inc. All rights reserved. Printed in Türkiye.
Saglik Gerecleri Tasarim San. Tic. Ltd. Sti., Zafer SB Mah. Gunduz Sk. No:13 E.S.B Gaziemir/Izmir/Türkiye
Lansinoh Laboratories UK Ltd
Lansinoh Laboratories Inc.
Lansinoh Laboratories Inc. Delaware Türkiye İstanbul Şubesi